# MICRO TOOLS KITAZATO®

Kitazato Corporation 100-10 Yanagishima, Fujf, Shizuoka 416-0932,Japan
Tel: +81-545-65-7122/Fax: +81-545-65-7128

Dibmed- Biomedical Supply S.L./C/Jorge Comín, 3 Bajo 1 2, 46015 Valencia, Spain
TEL: +34-96-305-63-95/FAX: +34-96-305-63-96

EC REP

Dibmed- Biomedical Supply S.L./C/Jorge Comín, 3 Bajo 1 2, 46015 Valencia, Spain
TEL: +34-96-305-63-95/FAX: +34-96-305-63-96

# CE 2797

### ENI MICRO TOOLS

**ICSI INJECTION PIPETTES:** Micro Tools for ICSI Injection (MTfNJL): 10 pcs/box.

**INDICATION FOR USE:** The Intracytoplasmatic Sperm Injection (MTfNJL) Injection Pipettes are used to inject a single sperm into oocyte.

**INSTRUKCIJS FOR USE:**

**Preparation:** · Make the drop of culture medium and PVP cover with Hypure Oil. Equilibrate it in the incubator for 15 minutes. · Set the pipette to suitable manipulation holder. **Note:** Recommend choosing the product whose inner diameter matches diameter of sperm in order to prevent a large amount injection of PVP. ·Place oocyte in culture medium, place motile sperm in PVP.

**Instrctions for use:** 1. Aspirate sperm tail first into MTfNJL. 2. Move the polar body at the 12 or 6 o'clock position, and hold oocyte with MTfHD. 3. Move sperm to tip of MTfNJL. 4. Penetrate oocyte with MTfNJL. **Note:** Make sure that the cell membrane is ruptured. 5. Inject single sperm into oocyte with minute amounts of PVP. 6. Gently withdraw MTfNJL. **HOLDING PIPETTES:** Micro Tools for ICSI Holding (MTHd): 10 pcs/box

**INDICATION FOR USE:** The Holding pipettes (MTHd) are used to hold the oocyte or embryo in position with the application of vacuum during the injection of a single sperm with the ICSI injection pipette, and other micromanipulation procedures.

**INSTRUKCIJS FOR USE:**

**Preparation:** · Set the pipette to suitable manipulation holder. **Instrctions for use:** 1. Hold oocyte or embryo with MTHd. 2. Keep holding it until the manipulation is finished.

**BIOPSY PIPETTES:** Micro Tools for Polar Body Biopsy (MTfBPPD): 10 pcs/box. Micro Tools for Blastomere Biopsy (MTfBBM): 10 pcs/box.

**INDICATION FOR USE:** The Polar Body Biopsy pipettes (MTfBPPD) are tools used in IVF/(ART) laboratories for removal of polar bodies from oocytes, which may be done in order to perform pre-implantation genetic diagnosis on the genetic material in the biopsied cell(s). The Blastomere Biopsy Pipettes (MTfBBM) are tools used in IVF/ART laboratories for removal of blastomere(s) form embryos, which may be done in order to perform pre- implantation genetic diagnosis on the genetic material in biopsied cell(s).

**INSTRUKCIJS FOR USE:**

**Preparation:** · Make the drop of culture medium, cover with paraffine oil. Equilibrate it in the incubator for 15 minutes. · Set the pipette to suitable manipulation holder. ·Place oocyte or embryo in culture medium.

**Instrctions for use:** 1. Open the zona pellucida, and then extract polar body from oocyte, blastomere from embryo or trophoctoderm from blastocyst. 2. Use the extracted cell and perform genetic diagnosis.

**PARTIAL ZONE DISSECTION PIPETTE:** Micro Tools for PZD (MTfPZD): 10 pcs/box.

**INDICATION FOR USE:** The Partial-Zone Dissection (MTfPZD) are used to cut an opening on the zone of an embryo mechanically in assisting embryo hatching prior to implantation.

**INSTRUKCIJS FOR USE:**

**Preparation:** · Make the drop of culture medium, cover with paraffine oil. Equilibrate it in the incubator for 15 minutes. · Set the pipette to suitable manipulation holder. ·Place embryo in culture medium.

**Instrctions for use (Cross dissection process of zona pellicida):** 1. Cut a part of zona pellicuda in the cross shap with MTfPZD. **Note:** Do not allow pipette to touch blastomere at this time. This could damage embryo.

**GENERAL INFORMATION: Raw Material:**Borosilicate glass.

**Quality Control Specification:** Each lot of Micro Tools receives the following tests· Endotoxins (≤0.5 EU/device (LAL)· Mouse Embryo Assay (2-cell MEA tests: ≥80%).

**Storage Instructions, and stability:** · Keep away from sunlight. ·Keep dry.

**Contraindications:** Note: Confirm specification of the product in the labeling prior use. ·To avoid deterioration, do not resterilize. ·Do not reuse. Reuse may cause contamination. ·Do not use if the package is opened or damaged. ·Do not use the product after expiration date or infected product and discard it(see the warning).

**Warnings:** · Aseptic technique should be used.

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