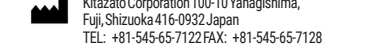
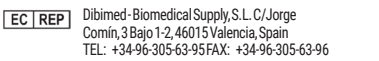
SEPASPERM® SOLUTION KITAZATO®

© 2011 KITAZATO CORPORATION. All Rights Reserved.

© 2011 KITAZATO CORPORATION. All Rights Reserved.

ENI SEPASPERM®

INTENDED USE: SepaSperm® Solution is a ready to use medium used for separation of motile sperm from seminal fluid with density gradient method.

COMPOSITION: SepaSperm® Solution medium contains: •Silane-coated silica particles •HEPES-buffered •EBSS (Earle’s balanced salt solution).

GENERAL INFORMATION: SepaSperm® Solution are available in bottles of 20ml, 50 ml and 100 ml.

INSTRUCTIONS FOR USE: CAUTION: The detailed usage of SepaSperm® Solution is determined based on each laboratory’s procedure and protocols that are optimized for individual medical purposes.

Preparation: • Allow the semen to liquefy at 37°C and initiate the procedure. • When gelatinous pellet is detected in the semen, leave it at room temperature for 30 more minutes or remove those using filters or other appropriate equipment. • Warm SepaSperm® Solution to 37°C prior to use. The products should be tightly capped when used in CO2 incubator to prevent pH alteration. • Properly mix the prepared density gradients by inverting the bottles several times.

Instructions for use: 1. Prepare a dual gradient system (45% – 90% or 40% – 80%) for each semen sample with a rinsing medium. • 90% gradient: Dilute SepaSperm® Solution with 10% of a rinsing medium. • 45% gradient: Dilute SepaSperm® Solution with 55% of a rinsing medium. NOTE: Gradients should be prepared and repacked under sterile conditions (e.g. LAF bench ISO class 5 and sterile recipient). For optimal results: • Prepare the gradients maximum 24 hours prior to use and store at 2-8°C and warm the gradients to room temperature or 37°C done hour before use. • Prepare the gradients needed for one day in one time and mix well. Discard excess unused prepared gradient. 2. Carefully place 2.5 ml of 90% density gradient on the bottom of the centrifuge tube. 3. Add 2.5 ml of 45% density gradient on top of the 90% density gradient. 4. Gently place the liquefied semen on the top layer (45% density gradient).

5. Centrifuge the tubes at 350 to 400 g for 15 to 18 minutes. If the pellets are not visible, centrifuge for a further 3 minutes. 6. Remove the supernatant.
QUALITY CONTROL SPECIFICATIONS: The following tests were performed for each lot of this product: •Sterility: SAL 10-3 •Endotoxin: < 0.5 EU/ml •Sperm Survival test: ≥ 80% survival after 4 hours exposure; ≥ 75% survival after 24 hours exposure. •pH: 7.20 – 7.90 (Lot release criteria: 7.20-7.60) •Osmolality: 300-330 mOsm/kg. All results are reported on Certificate of Analysis which is available upon request.
STORAGE INSTRUCTIONS AND STABILITY: • Store between 2-8°C. •Keep away from sunlight. •Once opened, store the product in the original packaging. •Once opened, the product is stable for at least 7 days under aseptic conditions and store between 2-8°C. Discard the remaining media if this period has passed. •The product is stable after transport (max. 5 days) at elevated temperatures (<37°C). •This product is stable until the expiration date labelled on the bottle if the packaging is unopened or undamaged.

CONTRAINDICATIONS: • Do not re-sterilize. • Do not freeze the product. • Do not use the product after expiration date. • Do not use if packaging is damaged or broken. • Do not use if product becomes discolored, cloudy or shows any evidence of microbial contamination.
WARNINGS: • Always works under strict hygienic conditions (e.g. LAF-bench ISO Class 5) to avoid contamination. Aseptic technique should be used - In case of eye or skin contact with SepaSperm® Solution, immediately flush eye/skin with water. • Observe all federal, state and local environmental regulations when discarding the product. • In case of infection, dispose the product appropriately in a prescribed manner. • The user shall be responsible for any problems caused by non-conformity to the present IUF. • This product is intended to be used by medical specialist trained in fertility treatment.

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