# KITAZATO

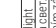
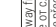
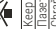
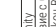
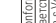
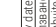
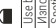
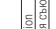
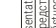
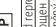
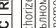
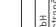
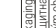
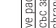
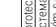
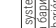
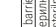
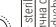
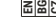
Copyright © KITAZATO CORPORATION All Rights Reserved. V15 2023-10

## CE 2797

## OPU NEEDLE

**Kitazato Corporation**  
100-10 Yanagishima, Fuji,  
Shizuoka 416-0922, Japan  
Tel: +81-545-65-7122 - Fax: +81-545-65-7118

**Dibimed - Biomedical Supply, S.L.**  
C/Jorge Comin, 3, Valencia, 46015 Spain  
Tel (+34) 963 056 395 Fax (+34) 963 056 396

	UK Representative	Importer
	US Representative	Importer
	FR Representative	Importer
	DE Representative	Importer
	ES Representative	Importer
	IT Representative	Importer
	JP Representative	Importer
	AU Representative	Importer
	CA Representative	Importer
	RU Representative	Importer
	CN Representative	Importer
	IN Representative	Importer
	KR Representative	Importer
	TW Representative	Importer
	TH Representative	Importer
	SG Representative	Importer
	MY Representative	Importer
	PH Representative	Importer
	VN Representative	Importer
	LA Representative	Importer
	KH Representative	Importer
	ID Representative	Importer
	MM Representative	Importer
	BN Representative	Importer
	TL Representative	Importer
	PG Representative	Importer
	FJ Representative	Importer
	PW Representative	Importer
	SB Representative	Importer
	TO Representative	Importer
	TV Representative	Importer
	UM Representative	Importer
	WF Representative	Importer
	CK Representative	Importer
	NZ Representative	Importer
	AU Representative	Importer
	PG Representative	Importer
	FJ Representative	Importer
	PW Representative	Importer
	SB Representative	Importer
	TL Representative	Importer
	PG Representative	Importer
	FJ Representative	Importer
	PW Representative	Importer
	SB Representative	Importer
	TL Representative	Importer
	PG Representative	Importer
	FJ Representative	Importer
	PW Representative	Importer
	SB Representative	Importer
	TL Representative	Importer
	PG Representative	Importer

### EN-OPU NEEDLE

**GENERAL INFORMATION** Refer to the label on the product or package before use. Read the instructions for use before use. BASIC UDI-DI: 4582231460PUL8. Target population: Patients undergoing ART procedures. For professional use only **INTENDED USE** The Kitazato OPU Needle is used for puncturing into human body and picking up tissue follicular fluid or oocyte from ovarian follicle.
**INSTRUCTIONS FOR USE 1.** Apply ultrasound probe to the puncture region and check the condition of the follicle. For transvaginal puncture, make sure the adjustment for the transvaginal ultrasound probe guide and the gauge of the needle. 2. Connect the needle hub or the female connector hookup to the syringe or the suction pump and make sure the direction to the follicle looking at the monitor of ultrasound. 3. Insert the needle tip to the follicle and aspirate the follicular fluid slowly. 4. For puncturing another follicle in the same ovary, keep the needle tip in the ovary. It is recommended to do flushing with the 10ml syringe accompanying the Double Lumen of OPU Needle. Conduct the flushing slowly. 5. Remove the needle after aspirating the follicular fluid. 6. Lightly press the puncture region with gauze. Check the condition of the patient for 15 to 30 minutes. 7. Immediately remove the vaginal speculum and continue to search oocytes from the aspirated. **PRODUCT SPECIFICATIONS - Sterility:** SAL 10-6 • LAL assay (Endotoxins): < 20 EU/device • MEA (1-cell, % blastocysts after 96h): ≥ 80%. MSDS and Certificate of Analysis are available upon request or can be downloaded from our website (kitazato-ivf.com). **STORAGE INTRUCTIONS** Store at room temperature • Use immediately after opening. **WARNINGS** • To avoid infection, do not re-sterile. • Do not use the product after expiration date. • Do not use if the package is opened or damaged. • This product is intended to be used by medical specialists trained in fertility treatment. • Aseptic technique should be used. • Observe all federal, state and local environmental regulations when discarding the product. • The user shall be responsible for any problems caused by incorrect use of the present IFU.
**PRECAUTIONS** • Confirm the specification (gauge and length) of the needle before opening the pouch. • Confirm adaptability with the other medical equipment to use together beforehand. • Pay close attention to the needle tip and avoid touching it touch until it is used for puncture as it might affect puncture efficiency. • Confirm the route of the puncture clearly before puncturing with the needle. • When clogging is confirmed in the needle or the connecting tube during tissue aspiration, remove the clog by applying pressure with the cannula attached to the product. However, avoid excessive force, and, depending on the situation, consider replacing the needle with a new one. • Confirm leakage through solution such as saline before use. • Do not use the product when air bubble is confirmed through a solution such as saline. • Discard immediately after use. • Pay enough attention to a false puncture. • Throw the product away promptly after having used it. Any serious incident (as defined in European Medical Device Regulation, 2017/745) that has occurred should be reported to Kitazato Corporation and, if applicable, the competent authority of the EU Member State in which the user and/or patient is established. **Note:** Instructions for use are electronically available at Kitazato-ivf.com or upon request.

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